


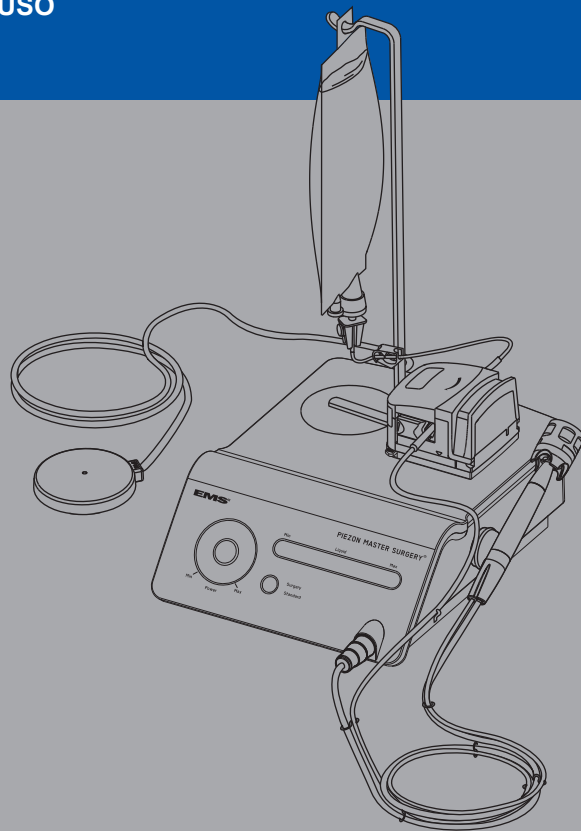
OPERATION INSTRUCTIONS
MODE D'EMPLOI
BEDIENUNGSANLEITUNG
INSTRUCCIONES DE EMPLEO
ISTRUZIONI PER L'USO
BRUKSANVISNING

Piezon Master Surgery®

ADDITIONAL SAFETY ADVICE FOR USA:
Caution! Federal (USA) law restricts this product to sale by or on the order of a dentist

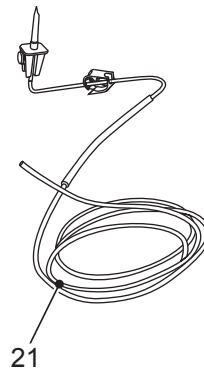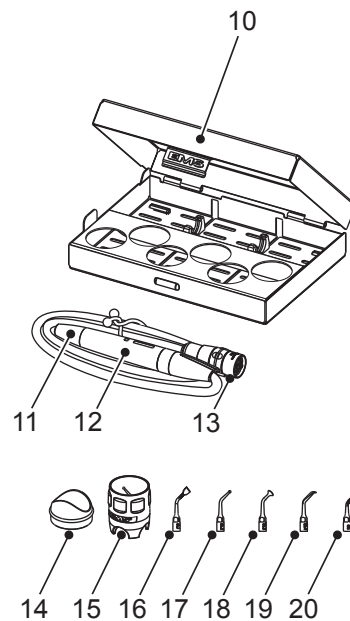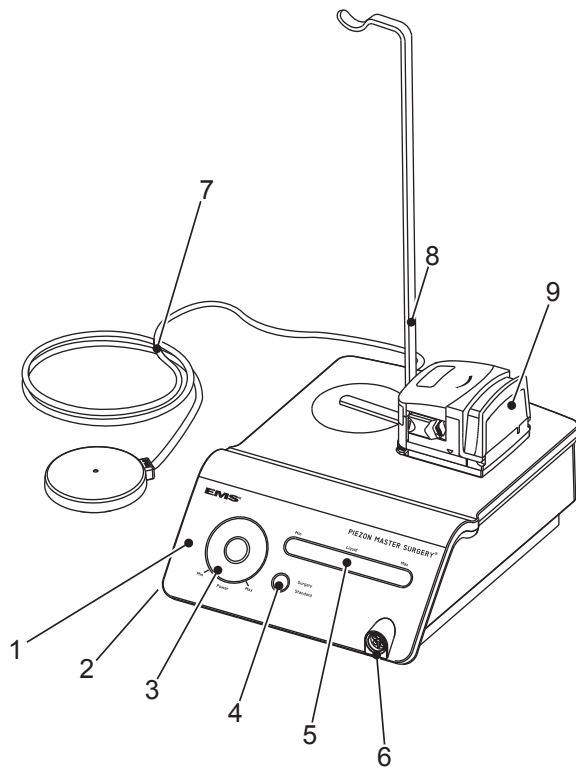EMS 
ELECTRO MEDICAL SYSTEMS

COMPONENTS AND CONTROLS

- 1 Control panel
- 2 Power switch (on left side)
- 3 Ultrasonic power adjustment
- 4 Working mode selection
- 5 Physiological solution flow rate adjustment
- 6 Connector for the handpiece cord
- 7 Foot pedal
- 8 Physiological solution hanger
- 9 Pump
- 10 SystemBox Piezon® ½ DIN
- 11 Handpiece cap
- 12 Handpiece
- 13 Handpiece cord
- 14 Magnetic handpiece support
- 15 Surgery CombiTorque®
- 16 Instrument SL1
- 17 Instrument SL2
- 18 Instrument SL3
- 19 Instrument SL4
- 20 Instrument SC
- 21 Sterile line

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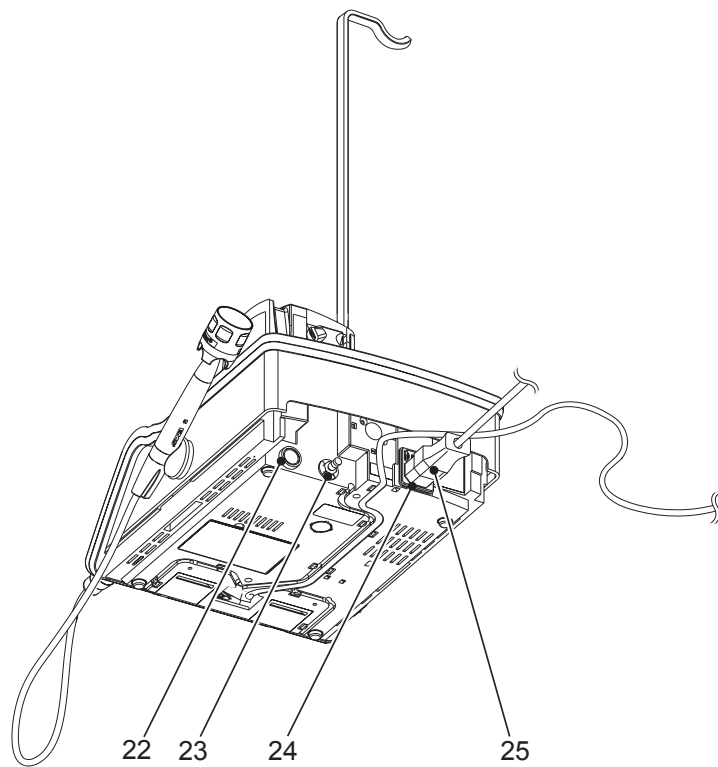
EMS supplies units with various accessories. The "Packing list" shows exactly what your unit includes. Note that the sterile line is delivered in a separate package.

Technical modifications

EMS reserves the right to modify the technique, accessories, operating instructions or contents of the set due to technical or scientific improvements.

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DEAR CUSTOMERS,

Congratulation for having purchased this new EMS product. It meets the highest quality and safety standards.

This ultrasonic unit is for use in surgical dentistry with EMS ultrasonic instruments.

EMS supplies ultrasonic instruments for the following indications:

- Implant site preparation
- Dental extraction
- Distraction
- Sinus lift
- Periodontal surgeries
- Cyst removal
- Extraction of bone bloc
- Bone harvesting
- Osteoplasty
- Osteotomy

For further details on the instruments and their respective applications, please refer to the instructions supplied with the Piezon Master Surgery systems or which are available from the distributor in your country.

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PLEASE READ THIS BEFORE BEGINNING WORK!

This operating instructions are to ensure the correct installation and use of this product.

Please read these operating instructions carefully as they explain all the most important details and procedures. Please pay special attention to the safety precautions.

Always keep this instruction close at hand.

To prevent injury to people and damage to property, please heed the corresponding directives. They are marked as follows:

Danger
Risk of injury

Caution
Risk of property damage or environmental harm

Please note
Useful additional information and hints

Prohibited

Authorized

About this manual

Note that the English version of this manual is the master from which translations derive. In case of any discrepancy, the binding version is the English text.

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We would be pleased to answer your questions or listen to your suggestions. We do, of course, provide support in case of technical problems. Please call directly your approved EMS repair center or your dealer.

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EMS

CONTENTS

Installation and connections.....	2
Foot switch connection	2
Electrical supply connection	4
Electrical safety precautions.....	6
Pump installation and removal	8
Solution hanger installation.....	10
Handpiece cord connection	12
Preparing for treatment.....	14
Handpiece installation.....	14
Sterile line installation	16
Operating the unit	20
Switching On	20
Standby mode	20
Working mode selection.....	22
Foot pedal function and backup switch	24
Regulating the flow rate	26
Regulating the ultrasonic power	28
Removing the sterile line.....	30
Cleaning, disinfecting, and sterilizing	32
Replacing the fuses.....	34
Safety precautions	36
Storing the product	40
Product Disposal.....	42
Warranty.....	42
Accessories.....	42
EMS-Service.....	44
Symbols	46
Technical data	52
Electromagnetic compatibility.....	56
Troubleshooting	58

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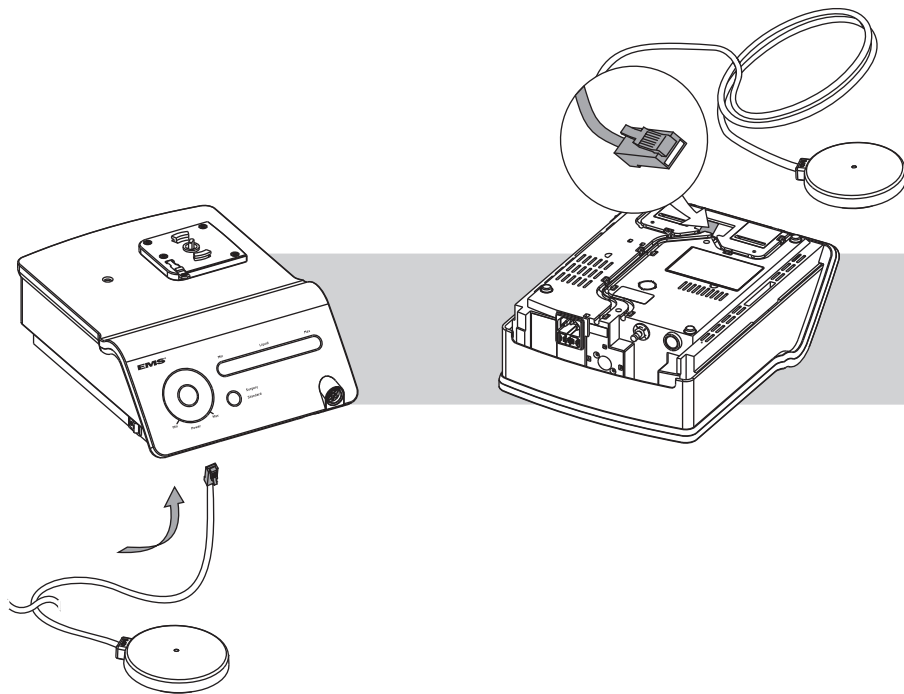
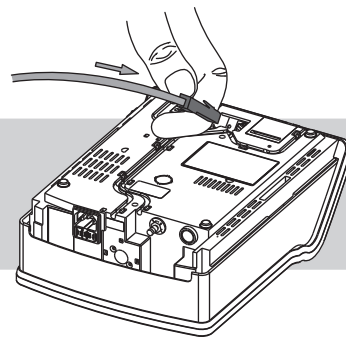
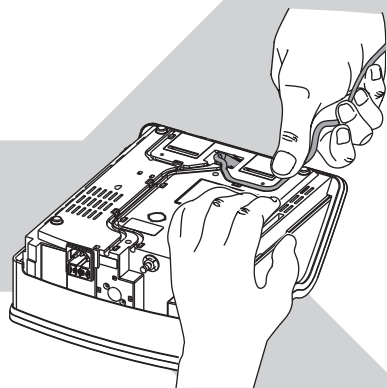
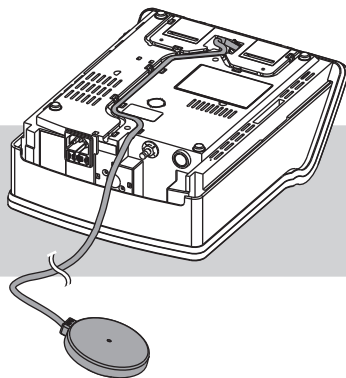
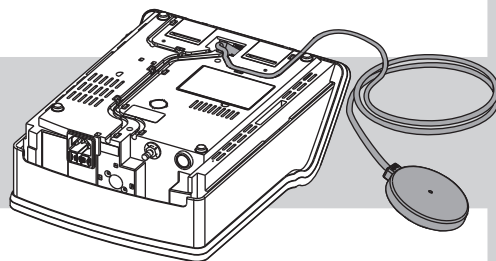
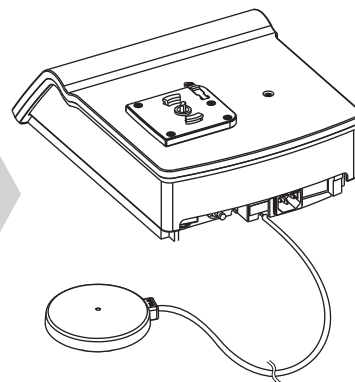
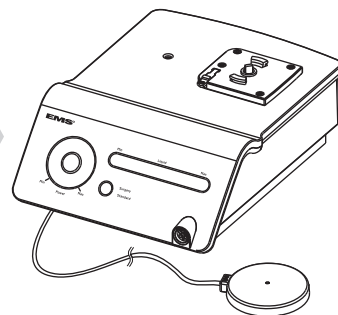
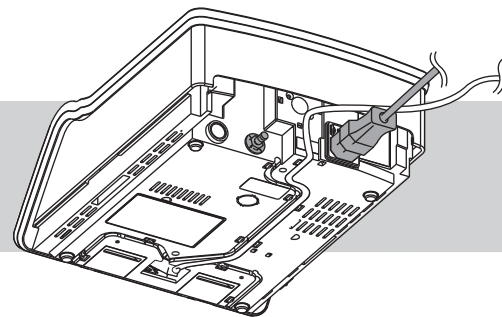
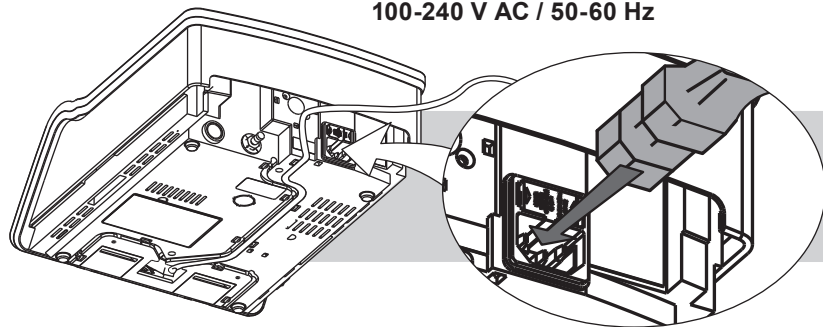
100-240 V AC / 50-60 Hz

👉 To prevent damage to the unit, check that its rated voltage corresponds to the local line voltage. Use the unit only in a FI protected mains supply (FI = Residual current protection).

👉 Only for USA or Canada: Connect only to Hospital Grade receptacle.

👉 When applicable connect the equipotential plug to other medical devices used.

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 Attach the sterile line with clips provided to housing and cord, with special care to route the sterile line away from the control panel.

 Before operation, activate irrigation at maximum flow for few seconds to make sure that all air has been removed from irrigation system.

 The sterile line is provided sterile by ethylene oxide gas (EtO). It is for single use only. Do not re-sterilize, do not re-use.

 To ensure sterility before use, inspect the sterile line and its package for any damage. Do not use in doubt that package seal has been broken, or that package or product is damaged.

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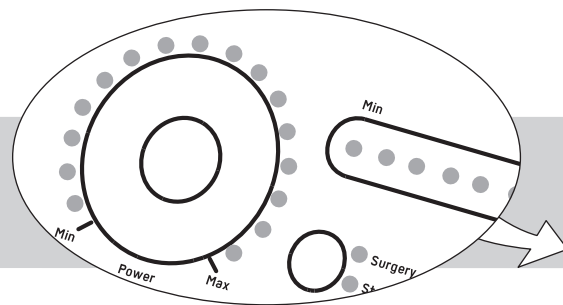
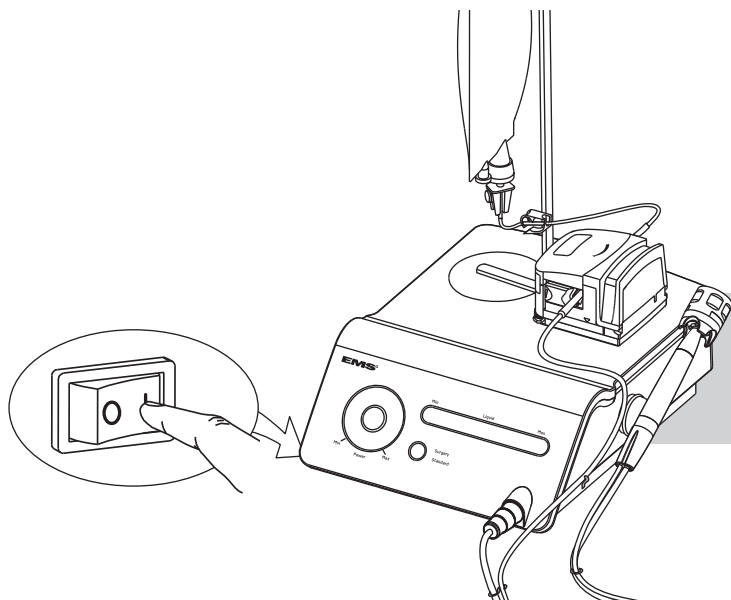
Einschalten

Standby mode

After 15 min. of inactivity, the unit turns itself in standby mode. Placing the finger on the sensitive zone wakes the unit off the standby mode. The unit returns to its previous configuration.

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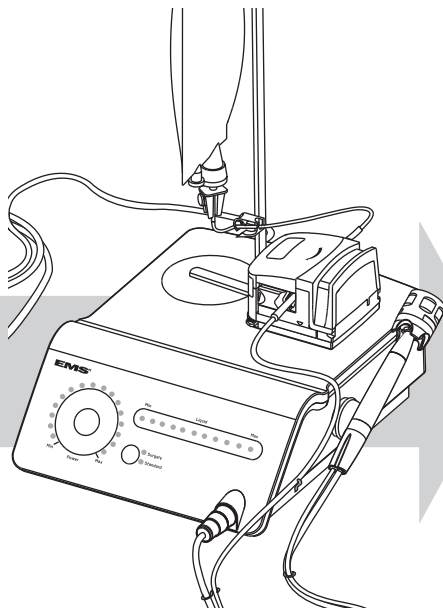
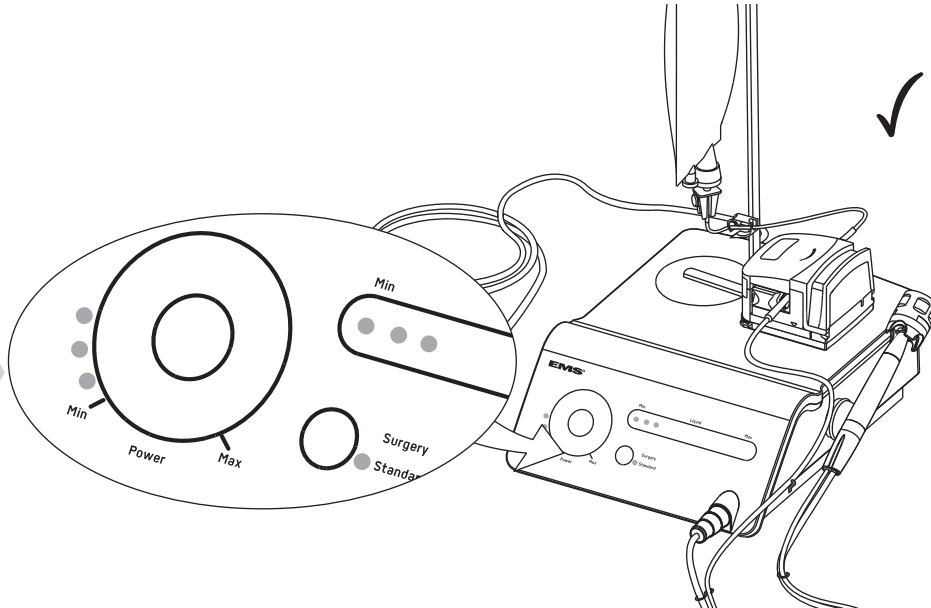
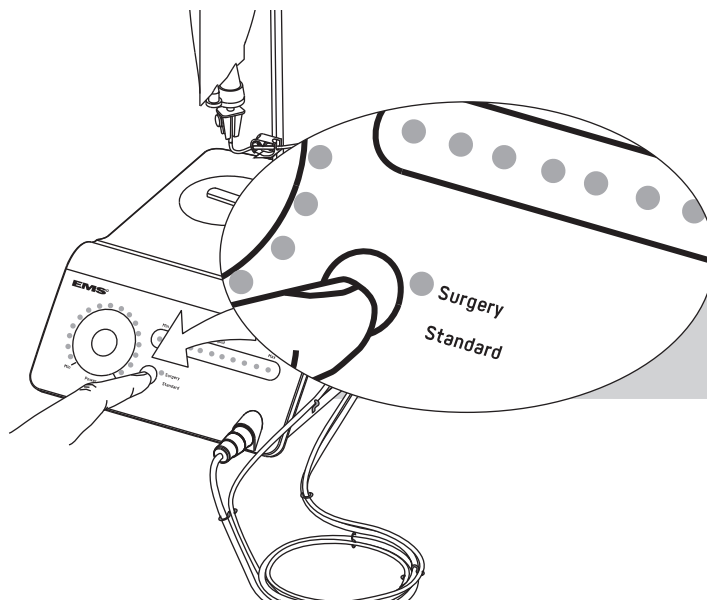
Working mode selection

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 Always select the "Surgery" mode when working on mineralized tissue.

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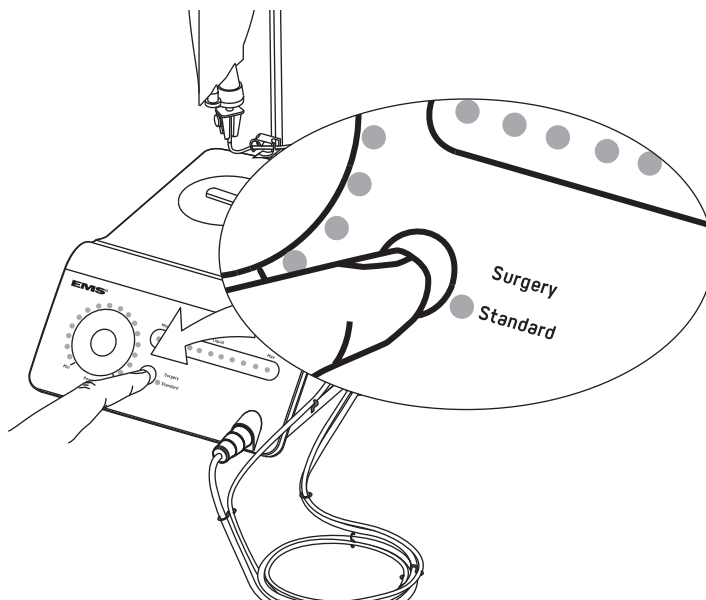
Foot pedal function and backup switch

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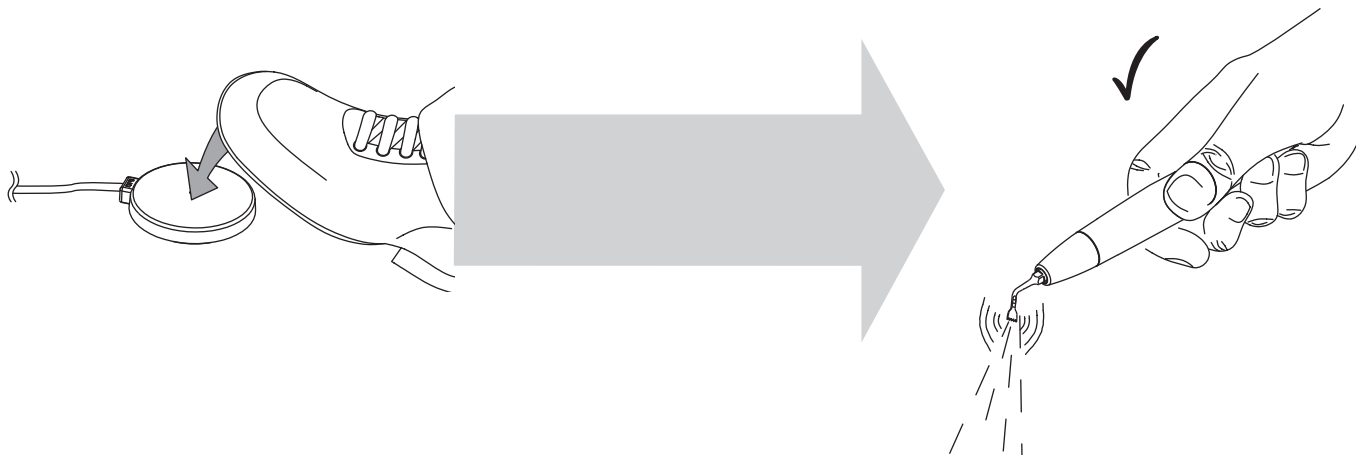
⚠ The backup switch must be used in the event of a foot pedal malfunction to allow finishing the treatment. It is highly recommended that the push button is held depressed by an assistant according to situation.

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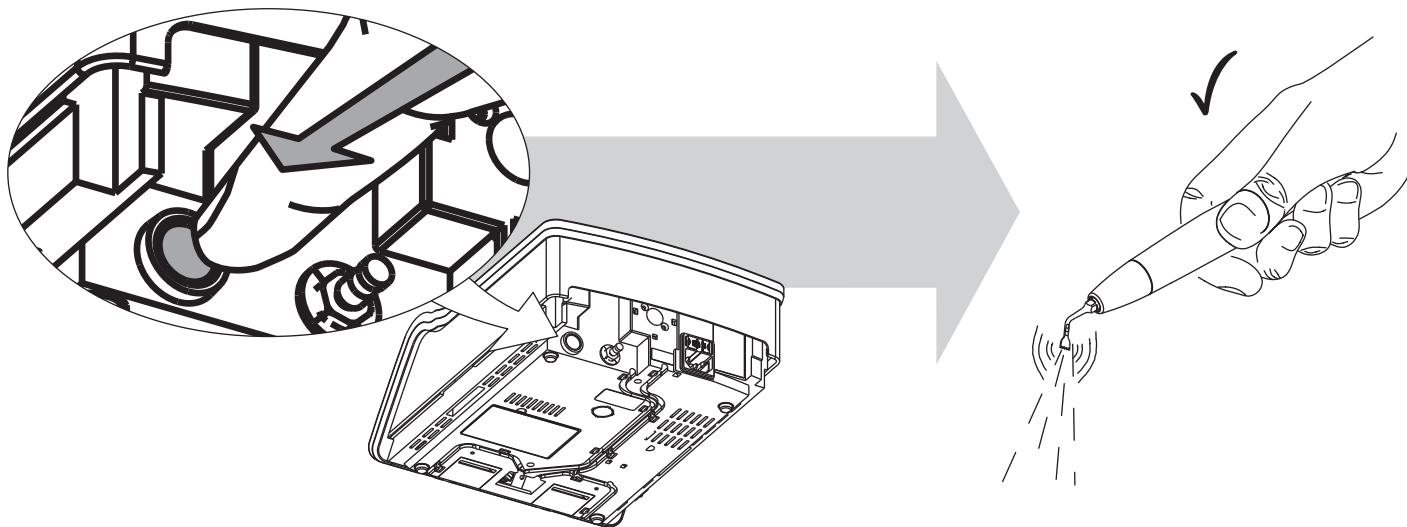
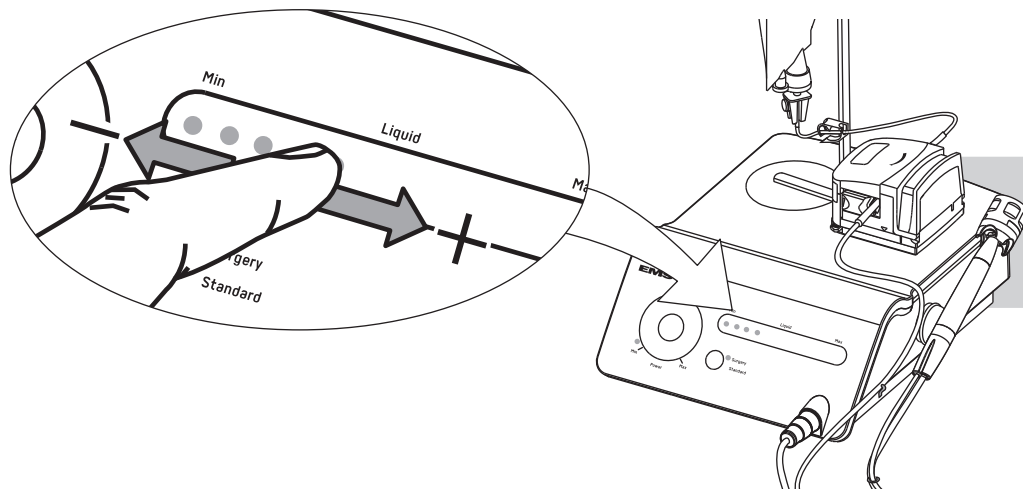
 The liquid flow rate is indicated in the operating instructions for Piezon Master Surgery systems which include precise details on the correct settings for every instrument.

 Never work dry with the instruments of your EMS Piezon Master Surgery. Always make sure there is enough irrigating liquid to avoid that the instrument and the tissues are subject to a potentially dangerous temperature increase.

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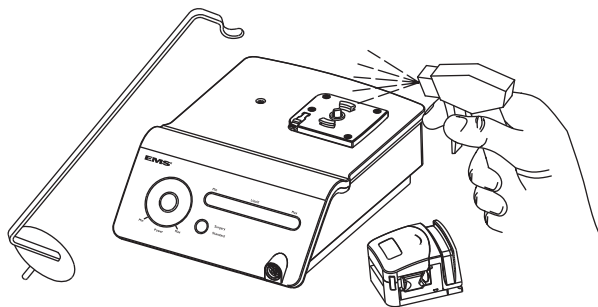
CLEANING, DISINFECTING, AND STERILIZING

Please conform to the recommendations of the Reprocessing Instructions manual delivered with your product regarding procedure of cleaning, disinfecting, sterilizing and packing of the components.

Please follow present-day rules about sterilization applicable in your country.

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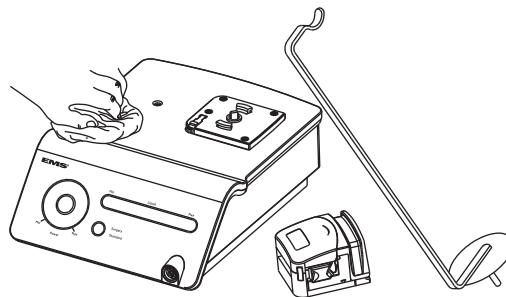
Unit

Clean the unit only with an alcohol-based, commercially available (ethanol, isopropanol), colorless disinfectant. The use of scouring powder or an abrasive sponge will damage its surface.

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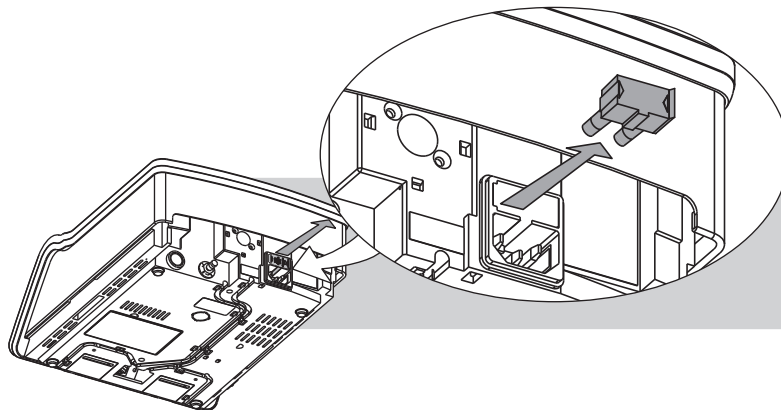
Apparaten kan inte steriliseras.

Replacing the fuses

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 Replace exclusively with the type of fuse indicated on the back of the unit.

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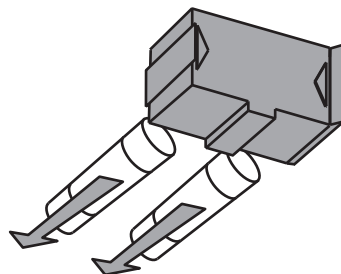
SAFETY PRECAUTIONS

EMS and the distributor of this product accept no liability for direct or consequential injury or damage resulting from improper use, arising in particular through non-observance of the operating instructions, or improper preparation and maintenance.

 **Contra indication:** ultrasonic oscillations may prevent cardiac pacemakers functioning properly. Therefore, we recommend that patients with a cardiac pacemaker should not be treated with this product.

 **Use for the intended purpose only:** Before using the product, make sure this operating instructions has been read. This also applies to any equipment used with this product. Failure to observe the operating instructions may result in the patient or user suffering serious injury or the product being damaged, possibly beyond repair.

 **Risk of explosion:** Do not use this product in the presence of inflammable anaesthetics or gases.

 **This product must be used only by trained and qualified personnel.**

 **Use only physiological liquids.** The choice of the liquids of treatment is under the responsibility for the user.

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Before use, always clean, disinfect, and sterilize the parts and accessories of this instrument. Please refer to the information provided in these operating instructions. Several parts that are non-sterile may cause bacterial or viral infections.

In extreme condition of use (60mn ON / 20 mn OFF / 40°C ambient) the handpiece could reach a temperature of 42°C which could induce a light discomfort to the patient.

Always examine the product for damage before commencing treatment. Damaged accessories or a damaged unit must not be used and must be replaced. Use original EMS spare parts and accessories only.

Have this product exclusively repaired by an approved EMS repair center.

Never use this unit on metal or porcelain restorations. The high frequency ultrasonic oscillations may loosen the restoration.

This product has been investigated with regard to safety from electrical shock and fire hazard. CSA (Canadian Standard Association) has not investigated other physiological effects.

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STORING THE PRODUCT

Keep the original packaging until the product is to be disposed of permanently. You can use it for shipping or storing at any time.

Should you wish to put your product out of use for an extended period of time:

- Proceed as described in the Cleaning, disinfecting, and sterilizing" instructions,
- Pack the product and all accessories in the original packaging.

Please refer to the storage and transport conditions in the "Technical data".

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PRODUCT DISPOSAL

The product, its accessories and its packaging do not contain any substances that are dangerous for the environment.

Disposal of this product must be handled according to local laws and regulations.

WARRANTY

The warranty is valid from the date of purchase of your product for the period indicated in the Registration Card.

Damages due to non-adherence to the operating instructions or wear out of parts are excluded from warranty.

The warranty of your product will be cancelled if you try to open it.

ACCESSORIES

Accessories are available from EMS or any authorized dealers. Please contact your customer service directly.

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EMS-SERVICE

Should your product need additional servicing or repairs, please send it to your dealer or our approved EMS repair center.

In the case of non-authorized repairs or damaged due to non-adherence to the operating instructions, EMS accepts no liability whatsoever. This will also void the warranty.

It is best to ship your product in the original packaging. It protects your product against damage during shipment.

Before dispatching your product including all accessories, please clean, disinfect and sterilize as described in the operating instructions.

When sending your product directly to the approved EMS repair center, please include the name of your dealer. This simplifies processing for us.

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TECHNICAL DATA

Description

Manufacturer	EMS SA, CH-1260 Nyon, Switzerland
Model	Piezon Master Surgery®
Classification EN 60601-1	- Class I - Applied part, Type BF - IP 20, unit - IP X1, foot pedal
Classification 93/42/EC	Class IIa
Mode	Continuous operation
Supply voltage	100-240 VAC 50/60Hz
Power consumption	105 VA
Fuses	T 1.6 A 250VAC (Ø5 x 20mm, time lag, low breaking capacity)
Ultrasonic specifications: - Maximum output - Frequency range	25 Watts 24-32 kHz
Weight (approx.)	2.9 kg (with pedal and hanger 3.4 kg)
Dimensions in mm (HxWxD)	161x 225 x 291 (H = 455 with hanger)
Operating conditions	+10° C to +40° C 30% to 75% relative humidity Altitude max. 3000 m
Storage and transport conditions	-10°C to +40°C 10% to 95% relative humidity 500 hPa to 1060 hPa air pressure

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ELECTROMAGNETIC COMPATIBILITY

 This product needs special precautions regarding electromagnetic compatibility and needs to be installed and put into service according to the note "Electromagnetic compatibility".

 Certain types of mobile telecommunication equipment could potentially interfere with this product. The separation distances recommended in the note "Electromagnetic compatibility" must be taken into account.

 This product should not be used adjacent to or stacked with an other unit. If adjacent or stacked use is necessary, this product should be observed to verify normal operation in the configuration in which it will be used.

 The use of accessories and cables other than those specified or sold by EMS as replacement parts, may result in increased emissions or decreased immunity of this product.

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TROUBLESHOOTING

Symptom	Solution
The unit does not start or stops	• Check the electrical connections • Check the fuses on back panel • Check main switch function: At least one LED must be visible on adjustments and working mode areas • The unit is in "stand-by" mode, tap on any sensitive zone to exit stand-by • Send the unit to your authorized EMS Service Center
The foot pedal does not activate the unit	• Check that the foot pedal is properly connected • Activate the pump with the back up switch on back panel • Replace the foot pedal • Send the unit to your authorized EMS Service Center
The unit is still activated when the foot pedal is released	• The foot pedal is active, check for jam at foot pedal • Disconnect the foot pedal and check function with the back up switch on back panel • The switch in the pedal is blocked. Turn the unit off at the power switch • Replace the foot pedal • Send the unit to your authorized EMS Service Center
Hand piece inert, or unstable function	• Check cord connection to the unit • Verify the power setting, increase power setting to maximum • Check that the instrument is tight on the handpiece using the combitorque • Install a new instrument on the handpiece • Replace the handpiece and/or the handpiece cord • Send the unit to your authorized EMS Service Center

cont. next page

Symptom	Solution
Liquid flow too low, or instrument tends to accumulate heat	• The pump does not function, see below • Remove the pump and check that drive shaft rotates smoothly when activated • Check that the silicon tubing is correctly placed in the pump • Check that the sterile line is the one recommended by EMS for this unit • Check that the sterile line is not damaged or obstructed • Check flow adjustment, increase flow to maximum • Check that the instrument orifice is not partially clogged up • Send the unit to your authorized EMS Service Center
Liquid flow too high	• Check flow adjustment and decrease it
The pump does not work	• Check that pump is correctly positioned on support • Check that the hand piece is correctly connected to the unit • Remove the pump and check if the drive shaft rotates • Check that the foot pedal is properly connected to unit • Activate the pump with the backup switch on back panel • Replace the foot pedal • Send the unit to your authorized EMS Service Center
The hand piece does not remain on the magnetic support	• Check the seal at the support base • Install the magnetic support on a metallic part of the unit • The support is demagnetized • Replace magnetic support
The hand piece cap does not remain tight	• Check for presence of the O-ring seal and replace if necessary • Check the thread on handpiece • Replace the handpiece

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EMS Italia S.r.l
EMS Corporation